

Introducing the efficient way to collect blood samples

New!

TEGOTM

Card

Collecting blood has never been this easy

Simple and safe

- Easy to apply blood samples to the blood card
- Blood sample cards can be inserted into envelope immediately after blood collection and are ready to be sent for laboratory testing
- Makes sampling process quick and more efficient

Compatible with existing lab processes

- TEGOTM is suitable for use in genetic/DNA testing such as genetic defects, trait selection, parentage and animal ID, as well as designated disease tests such as BVD and PRRS.
- DNA samples can be stored at room temperature for years without affecting sample integrity

Cost-effective and convenient

- Allows for shipment of blood samples through regular postal mail—refrigeration is unnecessary
- Room temperature storage reduces costs of handling and storing blood samples



TEGOTM products



New!


TEGO™ Card

Introducing the efficient way to collect blood samples


Simplifies the process, reduces the risks

TEGO™ was developed in close consultation with U.S. laboratories, veterinarians and livestock producers worldwide.


Quick and easy

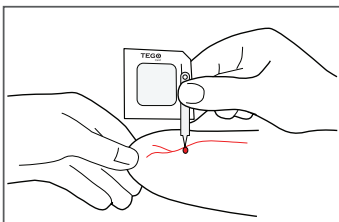


1. If blood is not readily available, use a new lancet to prick a vein on the animals ear or other appropriate location.

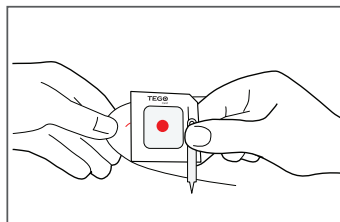

2. Holding the card by the frame, allow the blood to form a drop and blot onto the center area of the card.

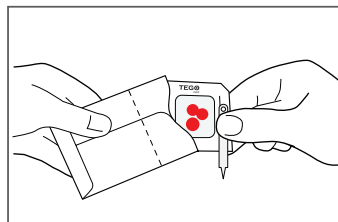

3. Place card into the envelope provided and label card and/or envelope as required.

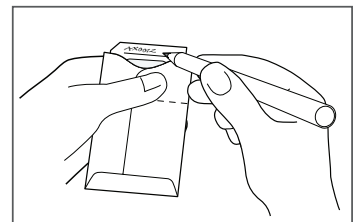

4. Fold over the open end of the sample envelope and place in a mailing envelope to send via postal mail. Safely dispose of the lancet or sharps device.

Available Blood Collection Kits

Product Code	Description	Qty/Pack	Qty/Carton
A100915	TEGO™ Card	1/pack	1,000/Carton
A100917	TEGO™ Card with Lancet	1/pack	1,000/Carton
A100919	TEGO™ Card	10/pack	1,000/Carton
A100920	TEGO™ Card with Lancet	10/pack	1,000/Carton

www.itlanimalhealthcare.com

For sales enquiries, please contact:

ITL Animal Healthcare

(The Americas, Europe, Japan, Korea)
Phone: 1-888-411-2851 (US Toll Free)
Phone: +1 (703) 435-6700
Fax: +1 (703) 435-6717

sales@itlanimalhealthcare.com

ITL Corporation

(Australia, NZ, Indonesia, Malaysia)
Phone: +61 2 9209 4004
Fax: +61 2 9319 3874
tego@itlcorporation.com

TEGO is a trademark of ITL Corporation, Melbourne, Australia.
©2011 ITL Corporation. All rights reserved. Patent pending.

There's security in knowing that you're providing the most advanced care for your animals

Single use only